

THE NOODLE CLASSIC AFO

INSTRUCTIONS FOR USE

WWW.KINETICRESEARCH.COM EMAIL: EMAIL@KINETICR.COM

THANK YOU FOR SELECTING THE NOODLE CLASSIC AFO CARBON FIBER BRACE!
PLEASE FOLLOW THE INSTRUCTIONS BELOW.


IMPORTANT! The Noodle Classic AFO should only be worn with shoes and socks

Donning the Noodle Classic AFO

Step 1: Remove the shoe's inserts and place the brace into the shoe. Place the shoe's inserts on top of The Noodle Classic AFO footplate.

Step 2: In a seated position insert your foot into the shoe and tie the shoestrings.


Step 3: Apply the velcro cuff strap on the calf.

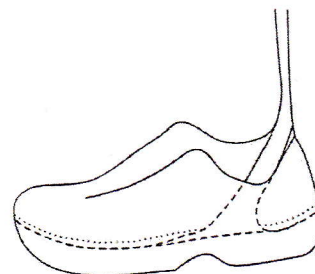

Doffing The Noodle AFO

Step 1: While seated unfasten the velcro strap. Untie and loosen the shoestrings.

Step 2: While seated and holding open the cuff, remove foot from shoe.

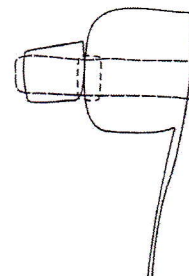

Caring for The Noodle Classic AFO

Cleaning: The cuff and straps may be hand washed with a mild detergent. Replacement pad packs may be purchased.

Supplemental Information:

Additional Information and instructional videos are available at www.KineticResearch.com. If you are experiencing problems with The Noodle Classic AFO contact the attending practitioner for assistance.