



If using an OP Flex Liner:
Apply a poly-adhesive glue on the brace. Let it dry. Continue to step 6.

Poly-adhesive glues are commonly used in orthotic work, brand name examples include 3M 4693 and SPS High Performance Poly-adhesive.

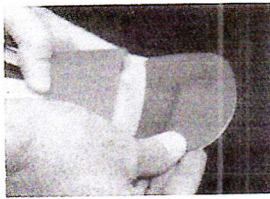


Remove the protective paper from the toe rise pad and attach the pad to the brace.

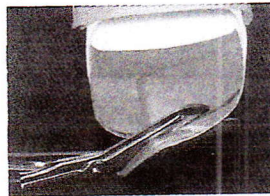

Trimming to Final Trim Line

Trim to the final trimming line along the toe shelf.

☐ If your toe pad includes a toe abduction strap, please continue to steps 8 – 10.

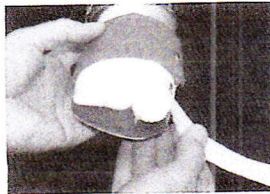

With the patient's foot back in the brace and straps secured, test the toe strap placement by finding your patient's best corrected toe position.



Attach the hook tab onto the bottom of the brace so that it aligns with the toe strap placement.



Pull the toe strap over the toe and attach it to the hook tab. Trim the strap at an angle to desired length.

For more information or questions, contact Customer Service:
call: **800.848.7332** or email customerservice@dafo.com

Cascade Dafo, Inc.

1360 Sunset Ave
Ferndale, WA 98248

ph: 800.848.7332
fax: 877.856.2160

intl: + 00 1 360 543 9306
www.cascadedrafo.com

DAFO Toe Pad and Strap Rev.03 (Mar 2014)