

Nanova Biomaterials, Inc. presents

# StarBright™

*5% Sodium Fluoride Varnish*

Designed with dental professionals in mind:

- Rigid tray for easy opening
- Varnish well for even greater convenience
- Enclosed, one-use brush for ease of use

With consideration for your patients:

- Dries in 10 seconds
- Fast fluoride release
- Sweetened with Xyitol
- Available in multiple flavors

Includes:

1 Mint

1 Strawberry

1 Bubblegum

(.40 mL each )

## Free Samples

# StarBright™

*5% Sodium Fluoride Varnish*

## Directions for Use

1. Clean teeth and remove obvious plaque and calculus from area to be treated. For best results, remove any excess moisture from the tooth surface.
2. Open the unit-dose of StarBright™ 5% Sodium Fluoride Varnish by gripping the rigid tray and slowly peeling back the foil cover, exposing the brush applicator and varnish well.
3. Remove applicator brush and thoroughly stir the StarBright™ 5% Sodium Fluoride Varnish in the well until the varnish appears to be uniformly mixed.
4. Paint a thin layer of varnish onto treatment area using broad and even strokes.
5. Let the varnish dry for approximately 10 seconds and then instruct the patient to close their mouth.
6. When dry, any excess varnish may appear “spotty” and may slide off the tooth surface. This will not affect the efficacy of the product as there is still a full layer of varnish on the tooth.
7. Dispose package and applicator after use.
8. Patient may leave immediately after treatment. Patient should be advised to not brush teeth for a minimum of 2 to 4 hrs.

Available in :  
Boxes of 30 or 150

To Order:  
Call (573) 875-6682

OR

Visit us online  
[www.nanovabio.com](http://www.nanovabio.com)

What CDT codes  
can be used when  
applying dental  
varnish?

D1206 - Topical  
application of fluoride  
varnish.

D9910 - Application  
of desensitizing  
medicament.

**NANOVA**