



# WrapStrap-R® Instructions

## EQUIPMENT

D-Ring WrapStrap-R®



Included	WrapStrap-R® (strap, micro-Velcro™ and D-Ring)
Required	Suspension sleeve
Optional	KISS® Valve (CMP19)

## STEPS

**1. Apply** the suspension sleeve to the socket.



**2. Lay** the WrapStrap-R® along the bottom edge of the sleeve. **Face** the micro-Velcro™ side of the WrapStrap-R® towards the sleeve.



**3. Wrap** the long end of the strap along the circumference of the socket. Feed it through the D-Ring. Ensure the micro-Velcro™ grips the sleeve. Do not let the sleeve extend below the strap.



**4. Feed** the remaining length of the long end of the strap through the D-Ring.



**5. Grip** both ends of the strap. **Ensure** the strap is flush with the end of the sleeve.



**6. Pull** tightly in opposite direction.



**7. Affix** both ends of the strap on top of the underlying strap.



**8. Smooth** out the strap and ensure a flush, sleek finish.

