


# Quick User Guide

## Part name of the main body

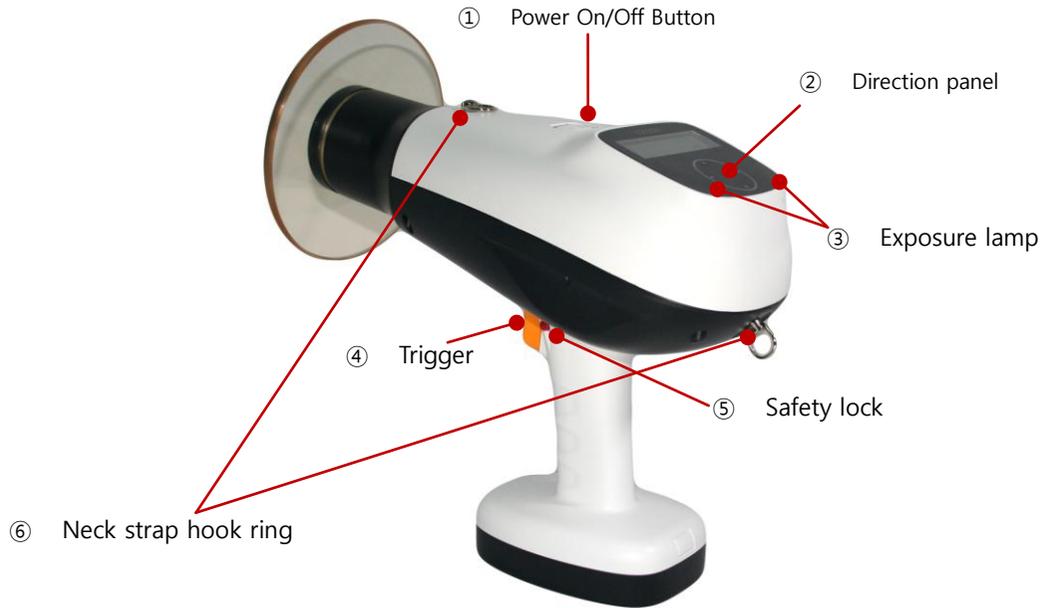

- ① Power ON/OFF button: Keep press power button till the device turn on or off (about 2 seconds).
- ② Direction panel:
  - ◀/▶ direction panel – select each function of X-ray exposure setting.
  - ▲/▼ direction panel – change a value of the function of X-ray exposure setting.
- ③ Exposure lamp: the lamp light is a green normally and it turns as orange on during X-ray exposure.
- ④ Neck strap hook ring: a hook for connecting Neck strap.
- ⑤ Trigger: X-ray exposure trigger.
- ⑥ Safety lock: safety option to avoid unintended X-ray exposure.

## How to use Cocoon

- ③ Keep press power button for 2 seconds to turn on the device.
- ② Select each function of X-ray exposure setting by touch ◀/▶ direction panel.



- ③ Set value of the function of X-ray exposure setting by ▲/▼ direction panel.
- ④ Unlock a safety lock after X-ray exposure setting. Lock- push lock button to right side, unlock – push lock button to left side.



- ⑥ When user pull the exposure trigger, a device status on GUI will changed as "ready" to "exposure" and LED lamp turned orange and "beep" sound occurs.
- ⑦ Keep press power button for 2 seconds to turn off the device.



# Quick User Guide

## How to charge the battery

- ① Connect adapter DC jack to Cocoon charging dock.



- ② Mount Cocoon on the charging dock as below.



- ③ The lamp of the charging dock and adapter is light on as red during the battery charging..



- ④ The lamp of the charging dock and adapter is light on as green when the battery fully charged.




## Main control screen consist



- ① Detector type: indicates detector type, PSP / Sensor / Film.  
② Patient type: indicates patient type, Adult or Pedo.  
③ Teeth type: indicates teeth type, bitewing, canine, molar, incisor.  
④ Device status: indicates the tube status.  
⑤ Exposure setting time: indicates exposure time.  
⑥ Battery status: indicates a battery status.

## Setting screen consist

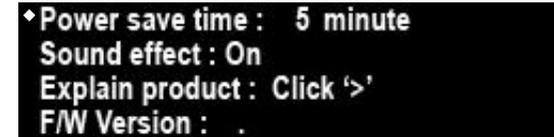

- ① Go to setting menu - Press ▲/▼ direction touch panel at the same time.
- Power save time: Device will turn off if the device not operates for a while. User can set the term.
  - Sound effect: Decide to sound when user touches direction panel.
  - Explain product: play a voice quick user guide instruction.
  - F/W Version: indicates a firmware version of the device.

# Quick User Guide

## How to use neckstrap

### Case I

- ② Put the external battery's output jack into the undercover of sleeve.



- ③ Hook the neckstrap hook to main body's upside ring.



- ⑤ Appearance after a neckstrap hooked with the device.



- ① Move the label side of the external battery to the magnetic part of the neckstrap, external battery will stick to the neckstrap.



- ④ Hook the other neckstrap hook to main body's downside ring. And then connect the external battery's DC jack to main body's connector.



- ⑥ Use the neckstrap as shown below.



### Case II

- ② Put the external battery's output jack into the undercover of sleeve.



- ④ Hook the neckstrap hook to main body's downside ring.



- ⑤ Appearance after a neckstrap hooked with the device.



- ① Move the label side of the external battery to the magnetic part of the neckstrap, external battery will stick to the neckstrap.



- ③ Hook the other neckstrap hook to main body's downside ring. And then connect the external battery's DC jack to main body's connector



- ⑥ Use the neckstrap as shown below.

