

# Preemie Pouch®

ALL-IN-ONE DRAINABLE OSTOMY  
POUCHES WITH SKIN BARRIER



# Preemie Pouch® Port


Designed for high, acid and runny output such as with an ileostomy, jejunostomy and duodenostomy.

Hydrocolloid and pouch come separated to make pouching easier.

Flexible, moldable, skin-friendly hydrocolloid.

Petals on daisy makes it easy to trim without cutting pouch.

Double thickness to reinforce around the stoma.

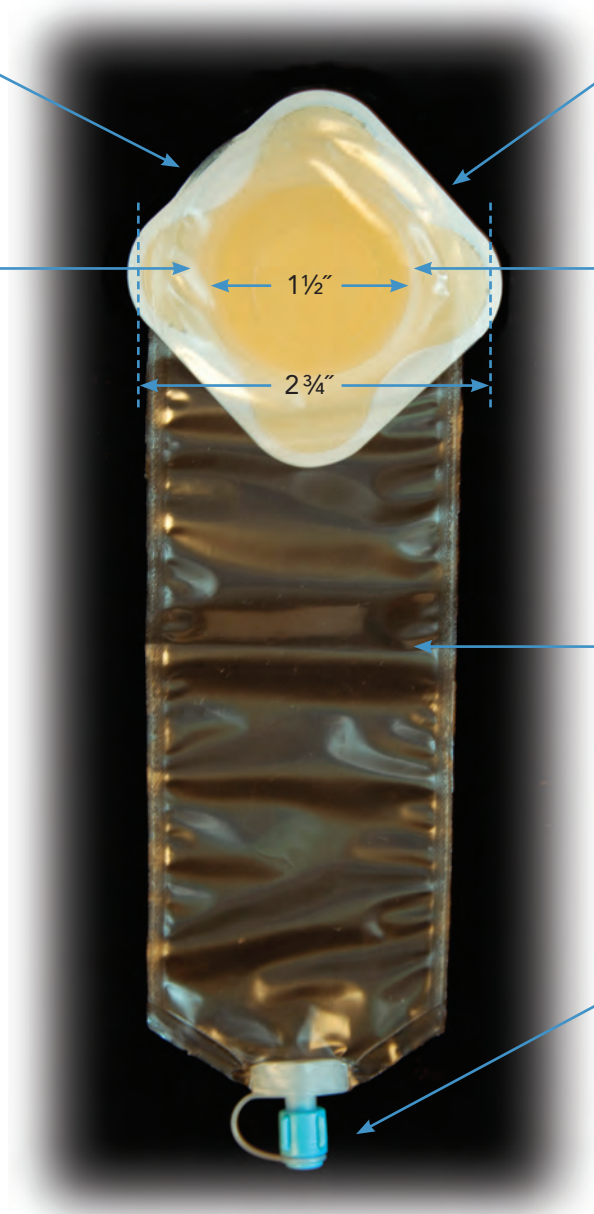

The pouch is free from plastic implements which makes it easy to empty without leaving residue.

Patented port system with luer-lock connection.

The hydrocolloid used on our pouches is also available in a 4" x 4" configuration:  
Neutra Skin Hydrocolloid  
7OS2432

**7OS2622S**  
For preemies > 800g

**7OS2612S**  
For preemies < 800g

**7OS2528S**  
For double barrel surgery



# Preemie Pouch® Open Drain

Designed for ileostomy and colostomy with thicker or formed stool that is difficult to remove through a port.

Hydrocolloid and pouch come separated to make pouching easier.

Flexible, moldable, skin-friendly hydrocolloid.

Petals on daisy makes it easy to trim without cutting pouch.

Double thickness to reinforce around the stoma.



The pouch is free from plastic implements which makes it easy to empty without leaving residue.

**7OS2012**  
For preemies < 800g



**7OS2422**  
For preemies > 800g

# Created For Your Special Needs



**7OS2528S**  
3½" and 8" pouch  
88 mm and 200 mm

For double barrel surgery with daisy shaped barrier and single thickness hydrocolloid.

Oval shaped cutting area with 8" (200 mm) pouch and exclusive drainage port for luer lock and luer slip syringe access. Largest opening available for our ostomy patients.



A stoma guide for sizing the stoma is included in each box of 10 pouches.

It is important to fold the barrier before using curved scissors to cut the hole shape. Closure device is supplied as well.

Make sure that the closure ends are bent away from the child.



**7OS2432**  
95 X 105 mm / Approximately 4" x 4"

Neutra Skin® is a pectin-based, latex free ultra thin barrier used whenever skin needs protection from adhesives. Neutra Skin® has been developed to produce the lowest possible skin irritation while maintaining superior adhesion. Neutra Skin® was specially made for neonatal use, it is flexible and gentle.

## HELPFUL HINTS

### ON THE COVER:

Chloe was born May 10, 2010 at 26 weeks. She weighed 15 ounces and was 11 inches. She spent 80 days in the NICU. Today she is a happy and healthy little girl.

- Before applying the pouch to the skin, be sure that the skin is clean, dry and free of any greasy substance that might impair adhesion.
- Warm the skin barrier between closed hands approximately 2 minutes and remove the protective barrier cover from the skin barrier.
- Carefully apply the pouch by maintaining gentle pressure approximately 2 minutes (see separate instructions for use).