

PULPDENT



Embrace Restoration & PFM Repair Kit

- ***All Surfaces***
- ***Fewer Steps***
- ***Saves Time and Money***
- ***Solves Big Problems . . .
Fast***



Embrace Restoration & PFM Repair Kit



*All Surface • Fewer Steps •
Saves Time and Money.*

- **Multi-functional kit: primes, protects, opaques, seals, finishes and polishes.**
- **Eliminates metal and ceramic primers, silane and bonding agents.**
- **Compatible with all restorative composites.**
- **No solvents, no modifiers, no mixing, no mess.**
- **Cures with all curing lights.**

PULPDENT

Embrace Restoration & PFM Repair Kit



➤ *Kit Contains 5 Products*

- **Embrace First-Coat™** bonds chemically and micromechanically to prepared ceramic and metal. Contains no solvents.
- **Embrace Seal-n-Shine™** penetrates and seals leaving a glaze-like finish on restored or repaired surfaces.
- **Embrace Opaquer** bonds chemically and micromechanically to metal, ceramic and tooth surfaces. 20-second light cure.
- **Kool-Dam™**, heatless liquid dam, protects soft tissue, teeth, and restorative surfaces. Remains flexible after curing. Easy to remove.
- **Porcelain Etch Gel**, 9.6% hydrofluoric acid, etches porcelain in 2 minutes.

PULPDENT

Embrace Restoration & PFM Repair Kit



➤ *Indications*

- **Repairing fractured porcelain restorations**
- **Repairing porcelain fractured off metal framework**
- **Bonding to exposed metal and ceramic (Embrace First-Coat)**
- **Opaquing exposed metal and discolored teeth (Embrace Opaquer)**
- **Etching ceramic (Porcelain Etch Gel)**
- **Protecting soft tissue, teeth and restorative surfaces (Kool-Dam)**
- **Sealing margins and microporosities in composite restorations (Seal-n-Shine)**
- **Penetrating surface finish and polish (Seal-n-Shine)**

PULPDENT



Embrace Restoration & PFM Repair Kit

Case Study

Courtesy of

Dr. Howard Glazer

PULPDENT

Embrace Restoration & PFM Repair Kit



1. Pretreatment photo of porcelain fractured off upper right cuspid on 7-unit bridge.



2. Place Pulpdent Kool-Dam to isolate and protect gingiva and adjacent porcelain teeth.

PULPDENT

Embrace Restoration & PFM Repair Kit



3. Micro-abrade metal with a micro-etcher, diamond or carbide rotary instrument.



4. Apply Pulpdent Porcelain Etch Gel (yellow) to porcelain for one minute.

PULPDENT

Embrace Restoration & PFM Repair Kit



5. Apply a thin layer of Embrace First-Coat, thin lightly with air, and light cure for 20 seconds.



6. Apply Embrace Opaquer to the metal surface with a brush applicator, and light cure for 20 seconds.

PULPDENT

Embrace Restoration & PFM Repair Kit



7. Apply your preferred composite and build up restoration. Contour, finish and polish the restoration.



8. Apply Embrace Seal-n-Shine and light cure for 20 seconds.

PULPDENT

Embrace Restoration & PFM Repair Kit



➤ Packaging and Ordering Codes

<u>Code</u>	<u>Description</u>
-------------	--------------------

EMPFM	Kit: 1.2 mL syringe each First-Coat, Seal-n-Shine, Opaquer, Porcelain Etch Gel, Kool Dam + accessories
EMFC	1.2 mL syringe Embrace First-Coat + 10 flocked tips
EMO2	1.2 mL syringe Embrace Opaquer, off-white shade

PULPDENT