

PULPDENT

Embrace™ Esthetic Opaquer Kit

20-Second Light Cure

Five Shades: Bleach White, Off-White, Light Yellow, Dark Yellow, Pink

Product Description

The Embrace Esthetic Opaquer Kit provides a palette of five opaquing shades for esthetic dentistry. These shades can be mixed together to produce a wide range of color variations. Use to mask out metals and discolored tooth surfaces for improved esthetics.

Embrace Opaquers enjoy all the advantages of Embrace resin technology. They behave favorably in the moist oral environment and form chemical bonds to metal, ceramic and tooth surfaces. The material spreads easily with a brush for fast application.

Embrace Opaquers cure with all lights. Curing time for a halogen light with a minimum of 300 mW/cm² is 20 seconds. More powerful lights will cure faster.

Note: Apply Embrace Opaquer to slightly moist tooth surfaces. Ceramic and metal surfaces should be dry prior to applying Embrace resins.

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to:

<http://www.fda.gov/MedicalDevices/ProductsandMedicalProcedures/DentalProducts/ucm404472.htm>

Instructions for Use

Ceramic and metal surfaces should be dry prior to applying Embrace Opaquer. Leave tooth surfaces slightly moist. Slightly moist tooth surfaces exhibit neither dryness nor pooled water. Lightly dry and remove excess water with compressed air or a cotton pellet. Tooth surfaces should be shiny or glossy. Overly wet tooth surfaces will result in decreased bond strength.

1. Prepare surfaces as follows:
 - a. Microabrade **metal and amalgam**
 - b. Microabrade **ceramic** or etch with Pulpdent Porcelain Etch Gel for two minutes carefully following instructions for use of etch gel.
 - c. It is not necessary to etch **dentin or cut enamel**.
2. If desired, apply Embrace First-Coat, Pulpdent DenTASTIC UNO or your preferred adhesive to the prepared surface. This step is optional.
3. Using a brush or applicator tip, apply Embrace Opaquer into the etched or abraded surface.
4. Light cure. Embrace Opaquer cures with all lights. Curing time for a halogen light with a minimum of 300 mW/cm² is 20 seconds. More powerful lights will cure faster.
5. Proceed with restoration using your preferred materials.

Caution

Uncured resin materials may cause eye or skin irritation on contact. Dental professionals should wear safety glasses and surgical gloves.

Storage and Handling

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.



CE
0459



PULPDENT Corporation

80 Oakland Street • Watertown, MA 02472 • U.S.A.
(617) 926-6666 or (800) 343-4342 • Fax (617) 926-6262
www.pulpdent.com • pulpdent@pulpdent.com

XP-EMOK-IN-04w
REV: 08/2017