

Use to convert non Velcro® compatible covered locking liners to allow for use with proximal KISS® Velcro® placards



CMP34/A Includes 2 KISS® Grips

Mark location of the center of the proximal socket exit on the liner

In a well ventilated area apply a coating of the newly formulated KISS® flexible adhesive (round bottle) to the non loop side of the KISS® Grip placard. Spread the adhesive using tongue blades.

Press into place on the liner with the proximal edge of the KISS® Grip Placard approximately 1 ½" superior to the marked dot on the liner. Hold the KISS® Grip firmly in place for 2 mins.

